NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact equipment supplier for further instruction.

SAFETY SUMMARY

WARNING

DO NOT use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions, and Instructions, contact equipment supplier before attempting to use this equipment, otherwise, injury or damage may result.

The best way to avoid problems related to pressure sores is to understand their causes and your role in a skin management program.

Your therapist and physician should be consulted if you have any questions regarding weight relief, self-examination of the skin, or individual limitations and needs.

Skin condition should be checked very frequently after the provision of any new cushion.

Make sure that ALL hardware are tight at ALL times.

PRODUCT DESCRIPTION

The Drop Base is designed to provide stable base of support for the individual who uses a seat cushion in the wheelchair. Using the Drop Base on certain chairs may allow the seat cushion to be recessed up to three (3) inches (7.5 cm). The angle of the Drop Base can be changed using the adjustable mounting clips. The Drop Base lowers seat cushion height for improved fit under tables or desks and improves foot propulsion.

The Drop Base is manufactured with universal crossbrace cut-outs which allows the Drop Base to fit most Invacare Manual (**Model DBA**) and Power Wheelchairs (**Model DBPA**).

Adjustable height mounting clips allow for changes of depth or angle in the field and also can be used for clinical evaluation.

Drop base mounting clips come in two (2) diameters, 7/8-inch (22mm) and 1-inch (25mm). These allow the base to interface with wheelchairs that have different seat rail diameters.

ASSEMBLY (FIGURE 1)

Remove the Drop Base, mounting clips and L-shaped brackets from the packaged container.

NOTE: The mounting clips can be positioned upward or downward depending on the amount of drop required.

Adjustable Mounting Clips



1. Attach the L-shaped brackets to the mounting clips using the hardware provided.
2. Use nuts and bolts provided and attach mounting clips/L-shaped bracket to Drop Base plate.
3. Place the Drop Base between the wheelchair seat rails with the cut-out of the Drop Base positioned over the crossbraces of the chair. Snap the Drop Base into place.

NOTE: The amount of drop available on each wheelchair varies depending on style of wheelchair (Hemi-height versus Standard height). On some wheelchairs, the crossframe may hit the Drop Base preventing maximum drop.

WARNING

Always test to see that the drop base is properly and securely locked in place before using.

Make certain ALL screws and nuts are tight at all times.

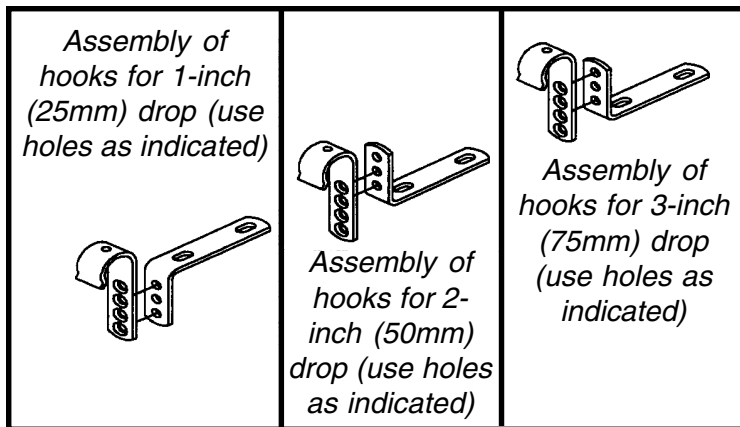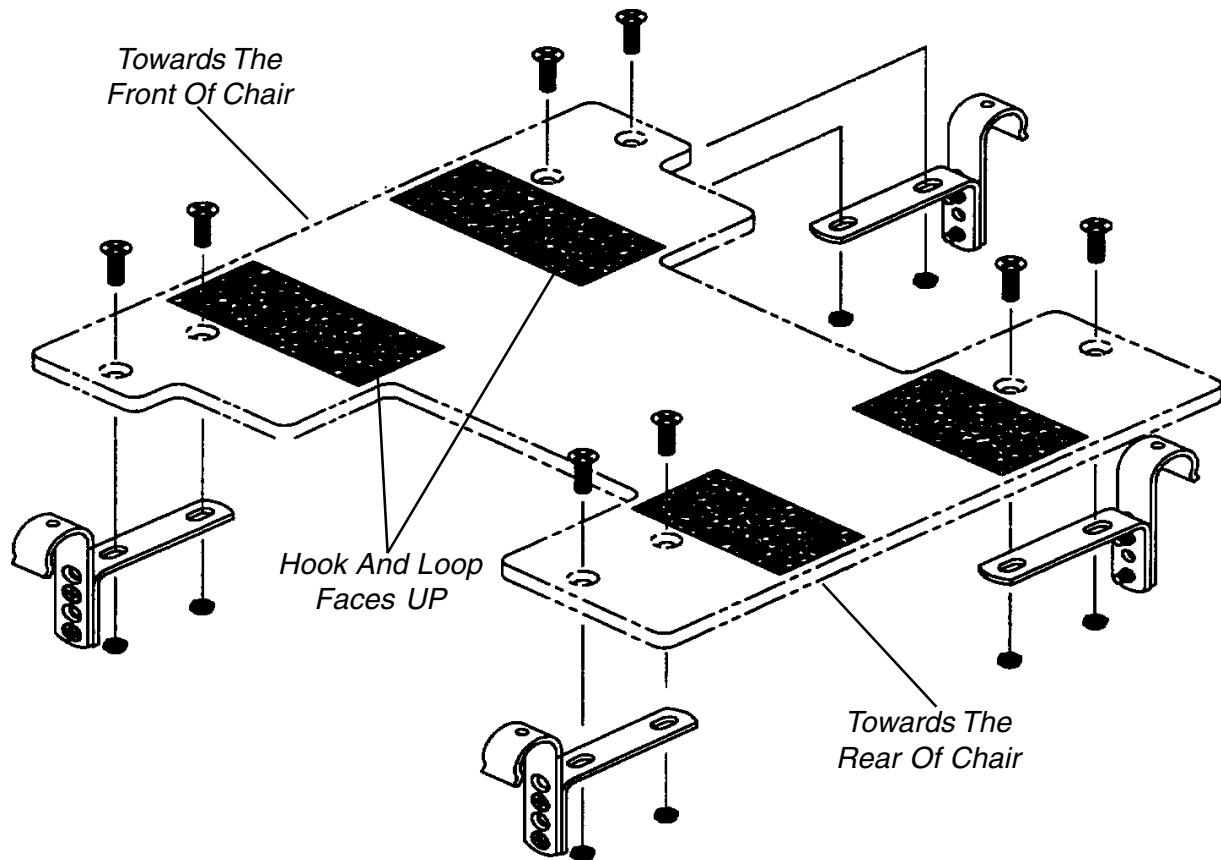Drop Base Assembly for 1-inch (25mm), 2-inch (50mm) and 3-inch (75mm) drop (use holes as indicated)

FIGURE 1 - ASSEMBLY

MAINTENANCE

These instructions MUST BE followed:

1. Clean weekly with warm water and mild soap solution. Wipe entire surface with a soft, clean cloth.
2. Visually inspect all parts weekly for deformation, corrosion, breakage, wear and/or compression.
3. Check hook and loop fasteners for adhesion and engagement.

WARNING

DO NOT continue to use this product if any such problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

UNITED KINGDOM LIMITED WARRANTY

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK warrants its product to be free from defects for a period of one (1) year from the date of purchase. With regard to the original purchaser/user only, Invacare UK warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period the product shall be proven to Invacare UK's / Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's / Invacare Service Representative option; Invacare UK's / Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department / Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK / Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK / Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE UK / INVACARE SERVICE REPRESENTATIVE.

UNITED STATES LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for a period of one (1) year. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you, do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealer's name, address, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE FOLLOWING INSTRUCTIONS.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE LAWS AND REQUIREMENTS.

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© 1999 Invacare Corporation
Invacare is a registered trademark of Invacare Corporation.
Invacare Corporation www.invacare.com